

ALLTION YSX ASSEMBLY GUIDE



ALLTION YSX ASSEMBLY GUIDE

1. Open the base box and remove foam.



2. Take the base out and place on the floor with wheels down. The base is very heavy, and requires a 2 person lift



3. Unscrew the base ring from the bottom pole.



4. Place the base pole fully into the base.




5. Rescrew the base ring onto the pole underneath the base.



6. The base of your Microscope is ready.



7. Unwind the screw on the top pole so it does not protrude inside the pole.

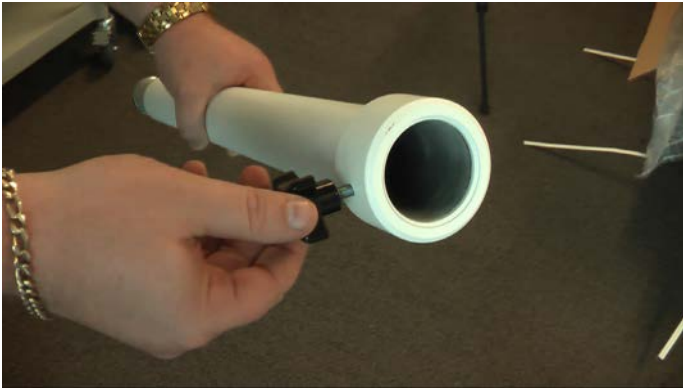

8. The screw should not protrude within the pole.



9. Place the top pole onto the base pole and screw tight.



10. Open the 2nd box with the microscope in it.



11. Remove the top layer to give access to all parts.



12. Place the power arm onto the pole and screw into place.






13. Take the microscope arm and feed the power cord through hole in the power arm.




14. Plug the power cord into the underside socket of the power arm.



15. Insert the guiding handles and lock into position.

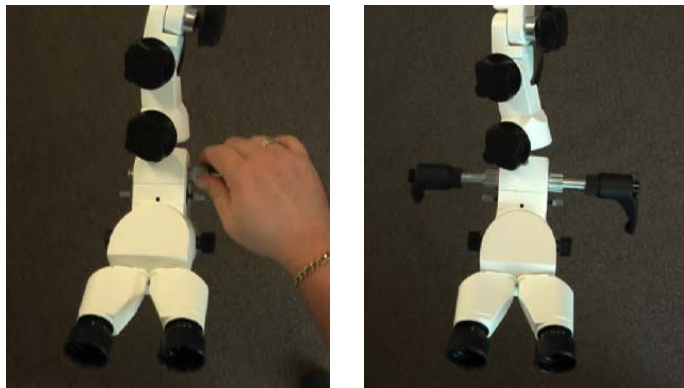

16. Use the provided screwdriver to twist the grub screw tight.

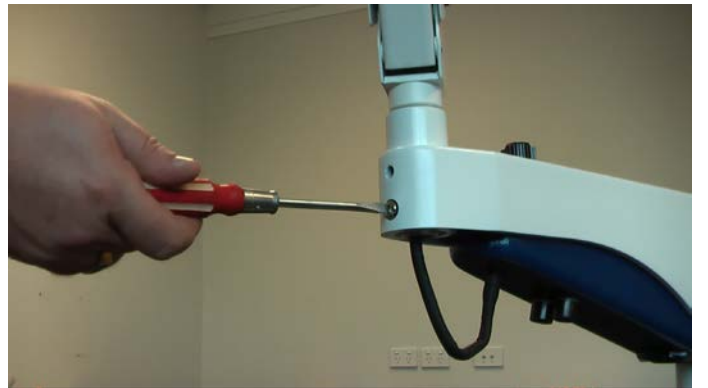

17. Insert and tighten final screw.



18. Plug the power cable from the wall into the power arm.

