


## HOW TO UPGRADE YOUR DEVICE FIRMWARE USING WINSPIROPRO

1. Turn on your PC and be sure the winspiroPRO software is closed.
2. Download the latest winspiroPRO software version via this link:  
<http://www.spirometry.com/download.asp?path=MIRwinspiroPRO/setup.zip>
3. Un-zip the "setup" folder that has been downloaded, then right-click on the file "setup.exe" and select "run as Administrator" to launch the Installation Wizard.

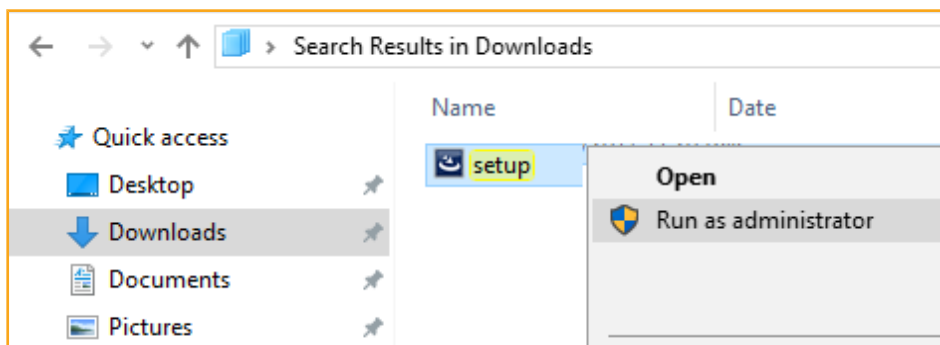


4. Follow the Wizard to install the software
5. Download the latest firmware version for your device from MIR website via this link:  
<https://www.spirometry.com/ENG/download/1Firmware.asp>

6. Look for your product on the website menu and click on download.

NOTE: in order to download the firmware, you have to login with the same email address and password you've used for the registration.


7. Once you click on download, the file should be saved on the "download" folder on your local disk.

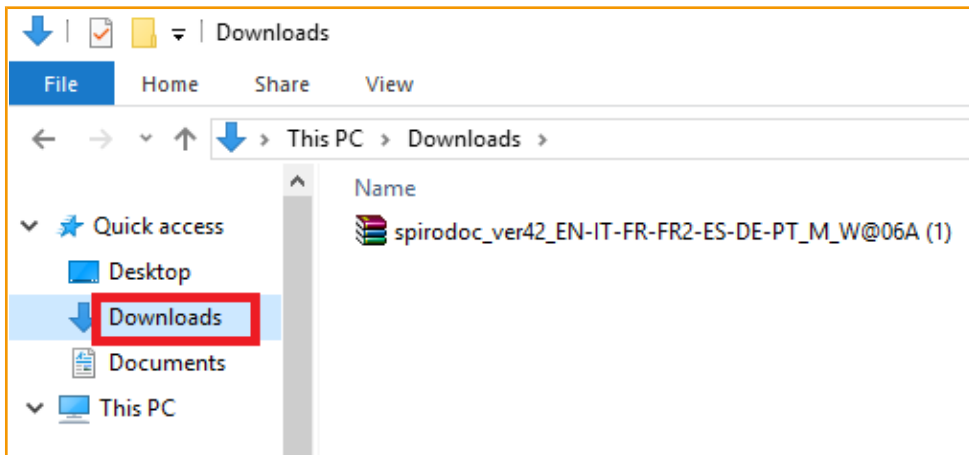


8. Unzip the file in the same folder and keep it ready for later

9. Open your winspiroPRO® software and connect your MIR spirometer to the PC via USB cable.

10. Click on Tools → Upgrade device internal software

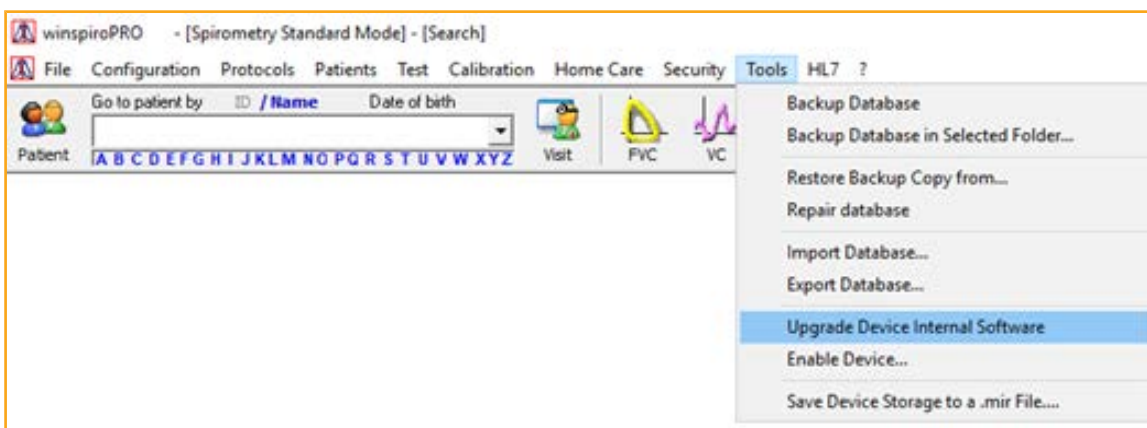


11. TEST your device is connected to your PC

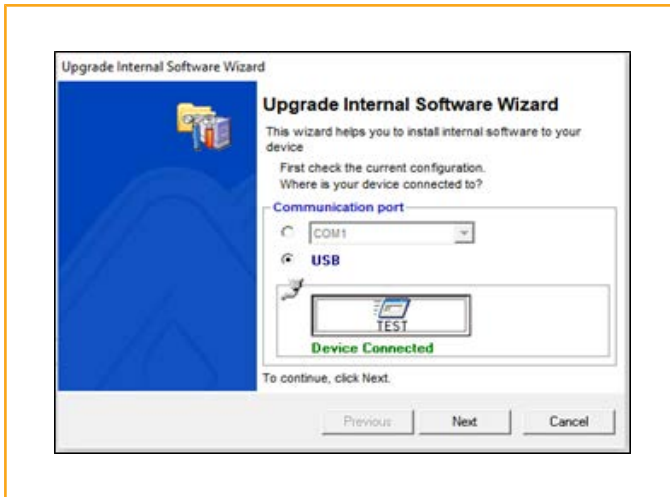


12. A green message Device Connected should appear, click NEXT

13. In the next window you can choose one out of different options.



Second option is the best choice: "Select manually the new version to install"  
so you can install the version you've just downloaded from spirometry.com  
click NEXT

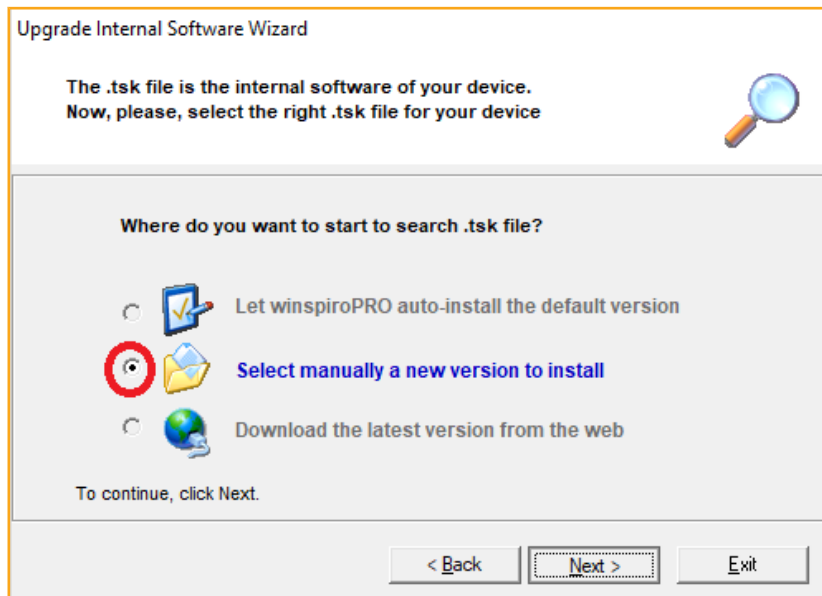

14. Go back to the file you've unzipped before which may be still in the "download" folder, and click on the firmware file.

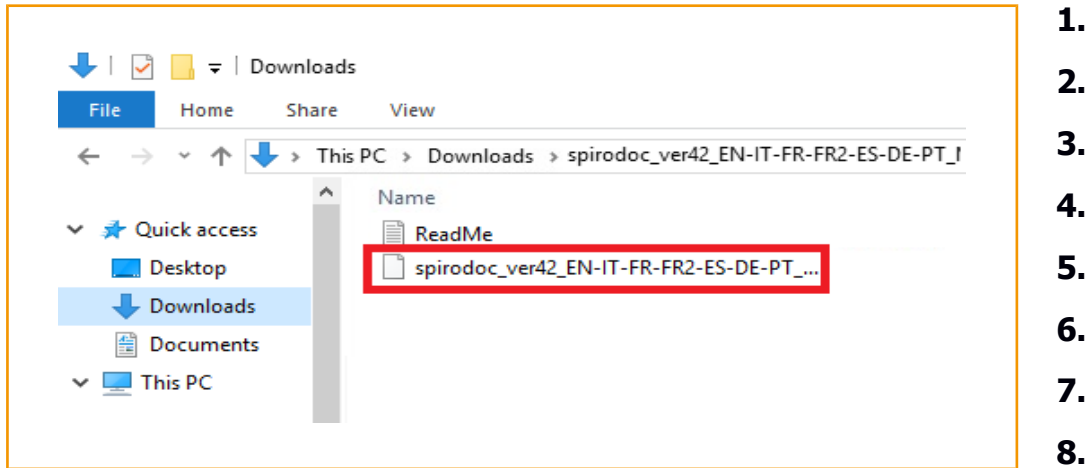

15. Follow the Wizard on winspiroPRO® and press Exit

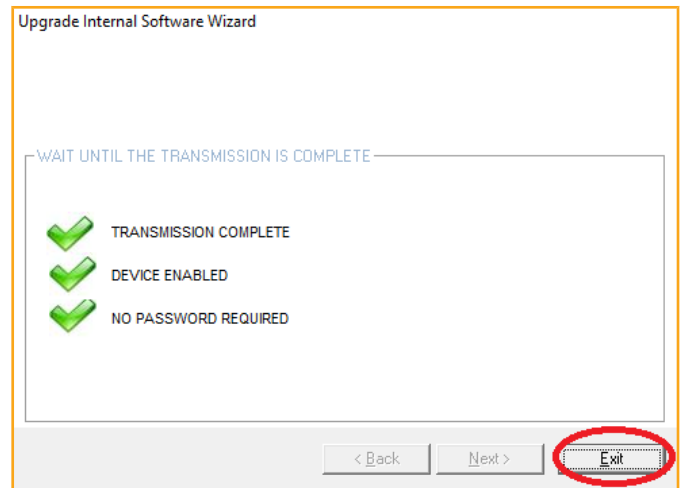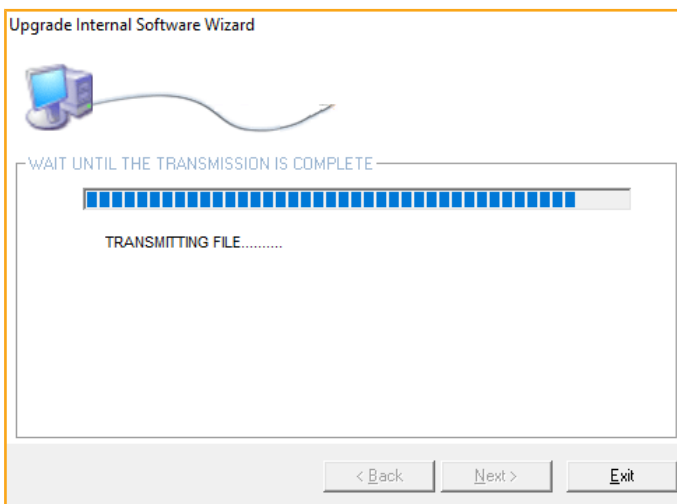

**MILESTONE: YOUR MIR EQUIPMENT IS NOW UP TO DATE!**